



1. six-sided bolt M6
2. fastening for connecting-rod
3. connecting-rod
4. lever
5. handle
6. gear casing
7. front wheel

B. MOVING FORWARD/BACKWARD

You choose the driving direction by putting the handle (5) on the gear casing (6) in the desired position (up = driving backward/down = driving forward). When the handle is in the central position, there is no drive and the wheelchair can be pushed.

Note : do not change the handle whilst driving.

C. STEERING

Steering is done by turning the grip on the lever (4). This makes the front wheel on the driving side of the wheelchair turn.

D. BRAKING

By pulling the lever (4) backward enough, the brake-block (8) on the driving side of the wheelchair will be pushed to the rear wheel and the wheelchair will stop.

If you have enough arm strength, you can push the lever forward again to move on. When you have only limited arm strength, you have to put the handle (5) in the central position first, then push the lever (4) forward and finally put the handle (5) in the desired driving position (for- or backward) again.

E. PARKING BRAKES

Your wheelchair is equipped with one-sided operating parking brakes on the rear wheels, which are connected to each other by a scissor system (9).

