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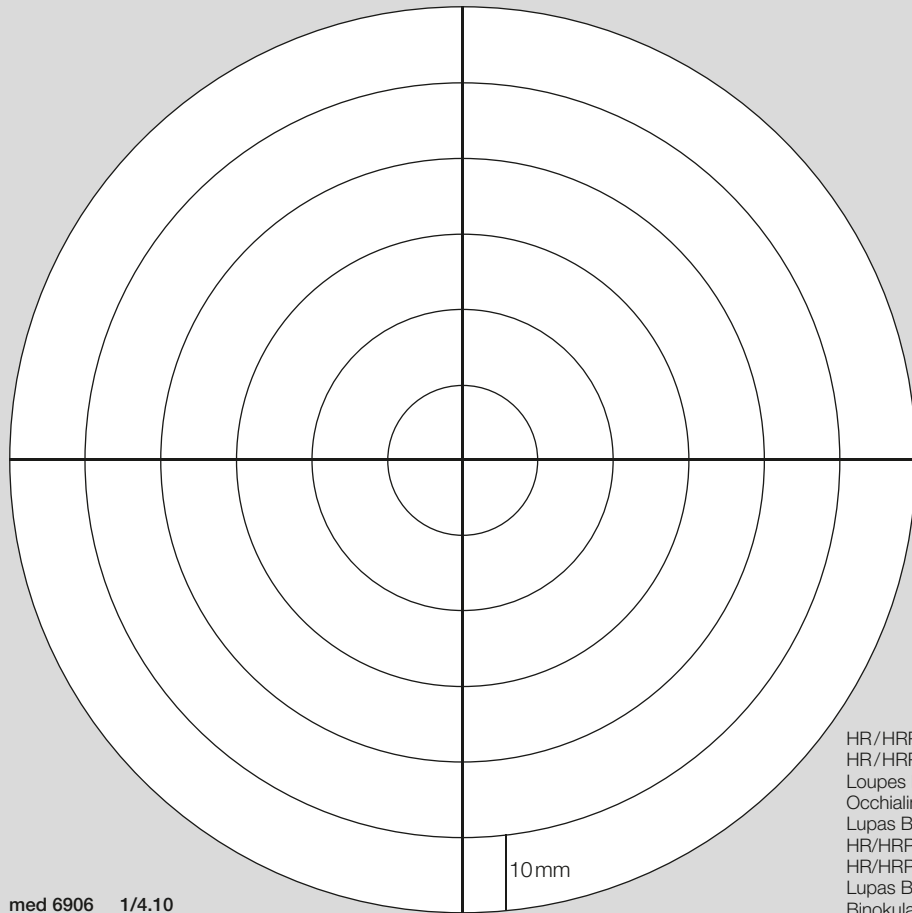
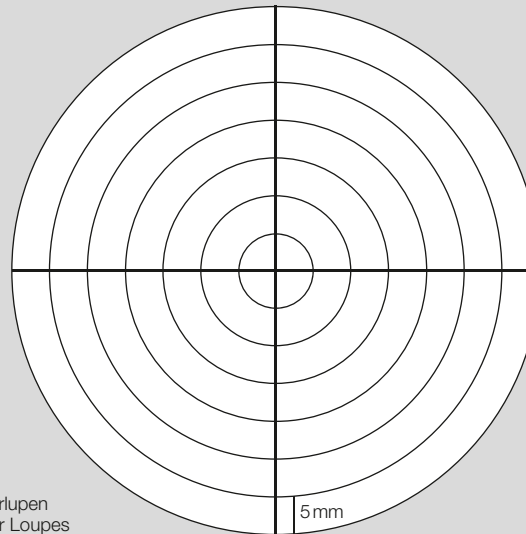
### ENGLISH

#### Using the target to set the proper Loupe Inter-Pupillary Distance.

Please read the following instructions prior to aligning your Loupe oculars.

- Loosen the Ocular screws on your Loupes.**  
HR/HRP Loupes – Grey screw knobs on the top of the Loupe frame.  
HR-C/C 2.3 – Small metal screw knobs on the top of the Loupe frame.
- Put your Loupes on.**
- Position the target at a distance from you equal to the working distance of your Loupe.**
- Begin by closing your left eye.**
  - With your left eye closed, slide the right ocular either left or right until the target circle fills your view through the loupe evenly and the cross hair of the target circle is in the middle of your view.
  - Staying still, now open your left eye and close your right eye, repeating this procedure.
  - Alternate closing each eye to ensure the target circle is centered through each ocular. Fine adjustments should be made to each ocular if the target circle is not completely centered.
- Open both eyes and confirm alignment.**  
Your view through the Loupes should be in one complete circle, no overlapping (like through a microscope or binoculars). If you see overlapping, please repeat steps 1 through 4
- Tighten the screws, locking in the PD.**
- Make note of these settings should the loupes ever be used by another individual.**

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